



Room Decontamination System 6110

Operators Guide





Tap here to access the operators guide.

Get Started



Help Introduction

<< Back

Below is the front page for the 6110 interface. You will see this screen on start-up.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List

Barcode Scan

Press the Get Started button to sign into the 6110.





User Sign In

[<< Back](#)

You are required to sign into the 6110 with a password assigned by your administrator.

[Help Introduction](#)

[User Sign In](#)

[Main Menu](#)

[Configuration / Run](#)

[Main Heads](#)

[Remote Heads](#)

[Save and Edit](#)

[Room List](#)

[Barcode Scan](#)

Enter your password using the number pad provided.

Press the Start button to complete your sign in.

7	8	9	CLEAR
4	5	6	CANCEL
1	2	3	RESET

Start **Cancel**

Note: If your password fails you will be notified. Please try to enter it again. If it continues to fail, please contact your administrator.



Main Menu

<< Back

After successfully signing into the AeroClave system you will be presented with the screen below.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List

Barcode Scan

To access the configuration of a room that has already been entered you may select from a list or scan the room's barcode, if this option has been chosen.

To create a new configuration.

Saved Rooms

Choose from list

Choose room configuration from saved list by name.

OR

Scan barcode now

Scan room barcode if this option has been implemented.

New Room

New configuration

Configure AeroClave to decontaminate a room not saved in the list.

Admin

AeroClave Config

www.AeroClave.com

Biodefense for the 21st Century

6110 general configuration to be accessed only by the AeroClave Administrator.



Configuration / Run

<< Back

The Configuration / Run screen is used to configure and run the 6110.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List

Barcode Scan

Name and save configuration for future use.

Choose heads and cubic feet on this page.

Emergency Stop Button appears after 6110 has been started.

After heads and cubic feet have been chosen, you may start the 6110 by pressing the Start Button.

ABCDEFGHijklmnopqrst + Sign Out ?

List Saved Rooms
Configure New Room

Main Heads ?

5 1 6

4 1234 2

Cubic Feet
Calculator

Select All Clear All

Remote Heads ?

5 1234 Calculator

6 1234 Calculator

Buzzer Off Strobe Off

AeroClave Title

Emergency Stop

START



Main Heads

<< Back

All heads chosen in the Main Heads section will run as a group for the time calculated from the cubic feet entered.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List



Barcode Scan

To enter the cubic feet of the space you may press the text box labeled Cubic Feet located in the middle of the main heads.

Main heads are labeled 1 through 4 and are positioned on top of the unit, with No. 1 located farthest away from the operator.

Heads 5 & 6 are hybrid ports located on the back of the unit and may be connected to remote heads.

The Calculator button presents a dialog to help you calculate the cubic footage of the space.

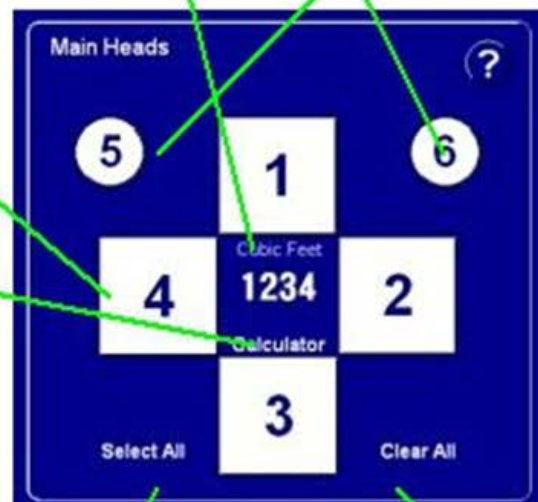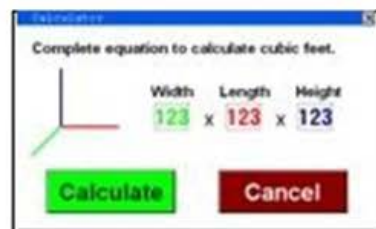

Press Select All button to choose all the heads, including the remote heads.

Clear All button deselects all the heads.



Remote Heads

<< Back

You may wish to run a remote head(s) independently from the main heads. An example might be a multi-room scenario where the main unit is in the larger space and the remote is in a smaller annex.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads


Remote Heads

Save and Edit

Room List


Barcode Scan

Select a head by pressing the corresponding numbered button.

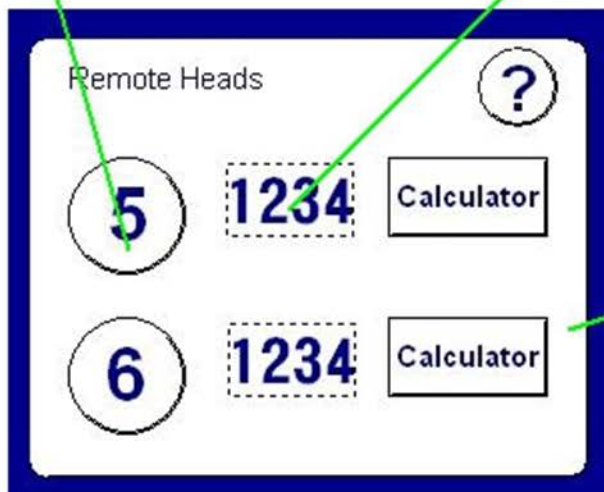

Enter the cubic feet of the space to be decontaminated by this remote head, either by clicking on the text box or using the calculator.

The Calculator button presents a dialog to help you calculate the cubic footage of the space.

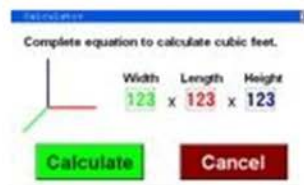



Save and Edit

<< Back

If you plan on decontaminating specific rooms on a regular basis, it may be helpful to save the configuration for future use. This is an optional function that will accommodate up to 100 rooms.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List

Barcode Scan

After completing the configuration, press the Add button to name and save.

Note: If you wish to clone a saved configuration, simply view the configuration and press the Add button to effectively do a Save As.

To edit the current configuration, press the Edit button to save.

Barcode is optional. You may either type the 6 digit number into the Barcode textbox or scan the barcode now.



Room List

<< Back

All saved configurations are listed sequentially.

Help Introduction

User Sign In

Main Menu

Configuration / Run

Main Heads

Remote Heads

Save and Edit

Room List

Barcode Scan

Please choose room from the list below. Sign Out

▼	ABCDEFGHIJKLMNQRST	▲
▼	ABCDEFGHIJKLMNQRST	▲
▼	ABCDEFGHIJKLMNQRST	
▼	ABCDEFGHIJKLMNQRST	
▼	ABCDEFGHIJKLMNQRST	
▼	ABCDEFGHIJKLMNQRST	
▼	ABCDEFGHIJKLMNQRST	▼
▼	ABCDEFGHIJKLMNQRST	▼
		+ Add New Room

Press the arrow button to select the corresponding room configuration.

Configure a new space by pressing the Add New Room button.



Barcode Scan

<< Back

Each room configuration may be assigned a specific barcode number for easy access.
*Optional functionality.

[Help Introduction](#)

[User Sign In](#)

[Main Menu](#)

[Configuration / Run](#)

[Main Heads](#)

[Remote Heads](#)

[Save and Edit](#)

[Room List](#)

[Barcode Scan](#)

You may access a room configuration by scanning a barcode from any screen.



Note: Barcode stickers are available from AeroClave.